

ZOLL®

AED Pro®

AED Pro® A-W

Operator's Guide



Before you begin

Be sure the defibrillation cable is plugged into the unit.

Procedure

To defibrillate the patient in semiautomatic mode:

Step	Action	Result
1	Press and release the On/Off button to turn on the unit.	On successful completion of the power-on self-test, the unit indicates the type of attached electrodes. If the defibrillation pads are not attached to the patient, the unit issues the voice and text prompt: <i>ATTACH DEFIB PADS TO PATIENT'S BARE CHEST</i>
2	If prompted, apply defibrillation electrode pads to the patient. (Refer to "Applying Defibrillation Electrode Pads" on page 3-2.)	When the pads are properly attached to the patient, the unit issues the voice and text message: <i>DON'T TOUCH PATIENT, ANALYZING</i> and then begins ECG analysis.
3	Allow the unit to analyze the patient's ECG.	The unit determines whether or not the patient has a shockable rhythm and then displays and voices its recommendation: <i>SHOCK ADVISED</i> or <i>NO SHOCK ADVISED</i>
4	After ECG analysis, which message appears? — If <i>SHOCK ADVISED</i> , continue with step 5. — If <i>NO SHOCK ADVISED</i> , go to step 8.	If a shock is needed, the defibrillator automatically begins charging. If a shock is not needed, the unit prompts you to perform CPR.
5	Wait for the defibrillator to charge.	When fully charged, the unit emits a charge-ready tone, repeatedly flashes the Shock button, and issues the voice and text prompts: <i>DON'T TOUCH PATIENT PRESS FLASHING SHOCK BUTTON</i>

